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## Product properties:

wound gel

KerraSol™ wound gel is a ready-to-use hydrogel for a virtually pain-free application. It keeps the wound moist, creates a protective film and thus promotes wound healing.

The application of KerraSol™ wound gel is nonirritating and virtually pain-free, with no restric-

tions to duration of product use.

KerraSol™ wound gel adds moisture to the wound, keeps the wound moist and forms an ideal wound healing environment to support the natural wound healing process.

The preservative (sodium hypochlorite) contained in KerraSol™ wound gel decontaminates a broad spectrum of bacteria (incl. pseudomonas aeruginosa, MRSA/MRE), viruses and fungi. Wound odour is quickly and reliably eliminated when using KerraSol™ wound gel.

#### Application:

The use of a wound spray or wound irrigation solution (e.g. KerraSol™ wound spray, Kerra-Sol™ wound irrigation solution) is recommended to remove exudate, fibrinoid residues and

other contamination.

Always generously cover the surface of the wound with KerraSol™ wound gel. Dress the wound with a suitable secondary dressing.

KerraSol™ wound gel dan be used in combination with compresses, plasters and other dressing material. There are no restrictions regarding the frequency and duration of use. The frequency with which the dressing is changed depends on the characteristics of the wound and must be adapted to the respective wound situation.

A slight chlorine odour indicates the product is working.

For hygienic reasons and to prevent contamination, KerraSol™ wound gel should only be used on a single patient.

Application restrictions:

There are no known side effects. KerraSol™ wound gel should not be used if there are known allergies to any of the ingredients. Do not use in combination with other wound cleaning sprays, solutions, gels and pre-saturated dressinas.

Interactions with other substances are not expected. During pregnancy and whilst breastfeeding KerraSol™ wound gel should be used with caution.

Do not use KerraSol™ wound gel on bleeding or exuding wounds.

## General safety instructions:

Only apply to external dermal wounds. Do not swallow, avoid eye contact. Keep away from children. Only use intact, undamaged tubes. Only use the products prior to the expiration date indicated on the tube. Once opened, product can only be used for a maximum of 12 weeks.

### Inaredients:

Water, sodium hypochlorite <0.06% (electrochemically activated mineral saline solution). Lithium magnesium sodium silicate

\*Issue 01/08 2015

Flaschen verwenden. Do not use if package is damaged

Crawford Healthcare GmbH Medizinpark 1 83626 Valley / Oberlaindern Tel: +49 (0)8024 46868-20 info@crawfordheathcare.com

www.crawfordhealthcare.com

Germany

Made in