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MPV MEDICAL

MicroDrop® Calimero 2

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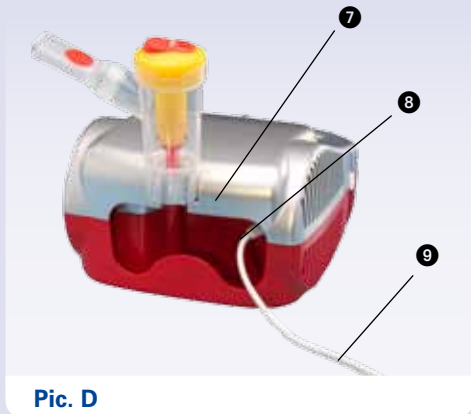
Professional inhalation for children 0 – 6 years

Congratulations on your purchase and thank you for choosing our unit. Our goal is to fully satisfy customers by offering them cutting-edge systems for the treatment of respiratory tract ailments. Carefully read these instructions and keep them for future reference. Use the unit only as described in this instruction manual. This is a medical device for home use and may only be used with medication prescribed or recommended by a doctor. Visit our Internet site www.mpvmedical.com to view the whole range of MPV MEDICAL products.

Indications

The inhaler MicroDrop® Calimero2 is used for treatment of respiratory diseases such as bronchitis, asthma etc., with an aerosol which is generated from inhalable medication prescribed by the physician.

Contraindications

Contraindications are as follows:

- Use in permanently ventilated patients by mechanical ventilation.

Complete equipment

- A1 Aerosol therapy unit (Compressor)
- A2 On/off switch
- B3 Air outlet port
- B4 Air filter
- C5 Carrying handle
- C6 Connection tube
- D7 Nebulizer holder
- D8 Cable compartment
- D9 Power cord

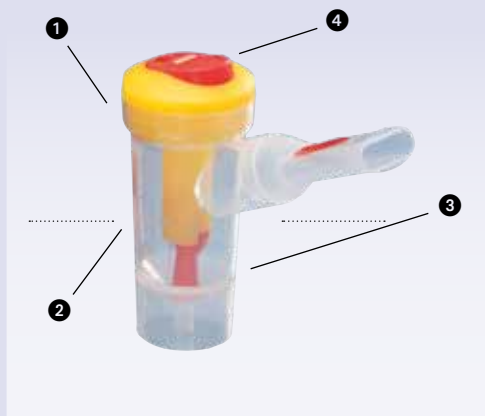
Pic. E: Complete equipment of the MicroDrop® Calimero 2

Accessory

- E1 Soft child mask
- E2 Baby angle / angle adapter
- E3 Mouthpiece with valve
- E4 Speed selector with valve system
- E5 RF7 plus nebulizer
- E6 Expiratory valve
- E7 Silicone mask size XS (0 – 12 month)
- E8 Silicone mask size S (1 – 3 years)
- E9 Silicone mask size M (3 – 6 years)
- E10 Compressor filter

Components of RF7 plus nebulizer for children

- F1 Upper part
- F2 Lower part
- F3 Complete nozzle
- F4 Speed selector with valve system

Pic. F: RF7 plus nebulizer for children

Important safeguards

Before using the unit for the first time, and periodically throughout its life, check the power supply cord for any damage. **If you detect any damage, do not plug it in. Take the appliance immediately to an authorised service centre or to your local dealer.**

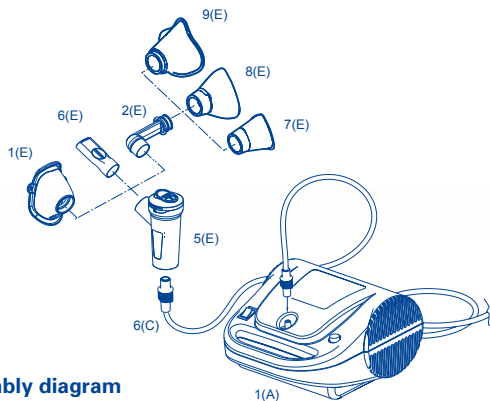
- The average lifespan of accessories is 1 year; however, we recommend you replace the nebulizer every 6 months in intensive use (or earlier, if the nebulizer is blocked) to always ensure maximum therapeutic effect.
- Children and disabled persons should always use the unit under strict supervision of an adult who has read this manual.
- Several parts of the device are small enough to be swallowed by children; therefore keep the device out of reach of children.
- Do not use the tubes and the cords of the device for purposes other than those specified as they may cause strangulation hazard; utmost attention must be paid to children and persons with special needs; usually, such persons are not capable of assessing the hazard correctly.
- The unit should not be used in the presence of anaesthetic mixture inflammable with air, oxygen or nitrous oxide.
- Keep the cord away from heated surfaces.
- Keep the power supply cord away from animals (for example rodents) as they may damage the insulation of the power supply cord.
- Do not handle the unit with wet hands. Do not use the device in humid environments (e.g. while taking a bath or shower). Do not immerse the device in water; should this accidentally occur, unplug the device immediately. Do not remove or touch the device immersed in water before pulling out the plug. Take the device immediately to an authorized service centre or to your local dealer.
- Only use the device in dust-free environments; otherwise, therapy may be compromised.
- The housing of this unit is not protected against the penetration of liquids. Do not wash the device under running water or submerge in water. Keep away from splashing water or other liquids.

- Do not expose the device to high temperatures.
- Do not put the device near heat sources, direct sunlight, or in hot environments.
- Do not obstruct or insert any objects in the filter or in its housing in the device.
- Do not obstruct the air vents placed on both sides of the unit.
- During use, always place the unit on a firm surface clear of obstacles.
- Check that there is no material obstructing the air vents before each use.
- Do not place any objects inside the air vents.
- Repairs must be performed by authorized personnel only. Any unauthorized repairs will void the warranty and may pose a safety hazard for the user.
- The average duration provided for compressor is: 1000 hours.

Do not modify this equipment without authorization of the manufacturer.

- The Manufacturer, the Retailer and the Importer are responsible for the safety, reliability and performance of the device only if: a) the device is used in accordance with the operating instructions b) the electrical system of the premises where the device is used is in compliance with current laws.
- **Interactions:** The materials used that come into contact with medications have been tested with a wide range of medications. However, due to the variety and continuous evolution of medications, the possibility of a chemical interaction cannot be excluded. We recommend that once the medication has been opened you should use it as soon as possible and avoid prolonged contact of the medication with the nebulizer.
- The manufacturer should be contacted for reporting problems and/or unexpected events related to device operation.

Assembly diagram

Instructions for use

Before each use, the nebulizer and accessories should be cleaned according to the instructions in the section “Cleaning, sanitization and disinfection”.

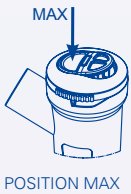
1. Insert the power cord (Pic. D9) into a power socket corresponding to the voltage of the device. This must be positioned so that it is not difficult to disconnect it from the mains.
2. Open the nebulizer by turning the upper part (Pic. F1) anticlockwise.
3. Pour the drug prescribed by the physician into the lower part (Pic. F2). Close the nebulizer by turning the upper part (Pic. F1) clockwise.
4. Connect the accessories as shown in the “Assembly diagram”. Sit comfortably, holding the nebulizer in your hands, place the mouthpiece over your mouth or use the mask.
5. If you use the face mask, fit it to the face as shown in the picture.
6. Start the unit by means of switch (Pic. A2) and breathe in and out deeply.
7. When the treatment is finished, turn off and unplug the unit.
8. It may happen that after a treatment session visible humidity deposits form in the tube (Pic. C6): disconnect the tube from the nebulizer and let it dry using the compressor air flow, as this prevents mould from growing in the tube.


A personal use of the nebulizer and its accessories is recommended in order to avoid any risks of infection.

How to use the "RF7 Dual Speed Plus" nebulizer with speed selector and valve

The nebulizer is professional fast and suitable for the administration of all types of drugs, including the most expensive ones, also in patients with chronic diseases. The geometry of the internal ducts of the RF7 Dual Speed Plus nebulizer ensures the ideal particle size providing effective treatment right down to the lower airways.

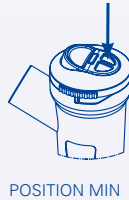To make inhalation therapy faster, press with your finger on the MAX marking on the speed selector with valve system (F4).

To make inhalation therapy more effective press with your finger on the speed selector with valve system (F4) opposite the MAX marking. In this way the valve systems in the nebulizer, in the mouthpiece and in the mask will ensure optimal medication delivery to the lower respiratory tract, limiting dispersion of medication into the environment.

Inhalation with mask

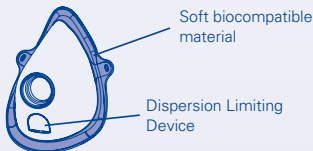
To make inhalation therapy faster, it is recommended to use the valve in position MAX for inhalation with mask.

Inhalation with mouth piece

To reach a maximal therapeutic effect, please put the nebulizer valve on position MIN and breath via the valve system.

SoftTouch Mask

SoftTouch masks have an outer edge made of soft biocompatible material that ensures excellent adherence to the face, and is also equipped with an innovative Dispersion Limiting Device. These distinctive elements that distinguish it allow greater sedimentation of medication in the patient and also limit dispersion.

During the inspiratory phase, the tab, acting as a Dispersion Limiting Device, bends inwards towards the mask.

During the expiratory phase, the tab, acting as a Dispersion Limiting Device, bends outwards from the mask.

Life cycle of the nebulizer

- The nebulizer is designed for multiple use. We recommend to replace all parts of the nebulizer, which could be in contact with the aerosol (C6, E1, E2, E3, E5, E7, E8, E9) after a year; twice a year for frequent use. The description of the cleaning of the nebulizer you find in the next section.
- When users change the exchange of the nebulizer (C6, E1, E2, E3, E5, E7, E8, E9) is mandatory.

Cleaning, sanitization and disinfection

Before undertaking any cleaning operation, switch off and unplug the device.

Cleaning

For unit and outer surface of the tube: Use only a damp cloth with an antibacterial detergent (non abrasive and solvent free). Before and after each use, sanitize the nebulizer and the accessories using one of the methods as described below. A personal use of the nebulizer and its accessories is recommended in order to prevent any risk of infection. Open the nebulizer by turning the upper part (Pic. F1) anticlockwise. Detach the nozzle from the upper part (Pic. F1) and remove it.

Sanitization

method A: Sanitize the accessories E1 (without elastic) E2, E3, E7, E8, E9, F1, F2, F3, F4 using warm water (40°C approx.) with mild (non-abrasive) washing-up liquid or wash them in a dishwasher, running a hot cycle.

method B: Sanitize the accessories E1 (without elastic) E2, E3, E7, E8, E9, F1, F2, F3, F4 by soaking them in a solution of 60 % water and 40 % white vinegar. Rinse thoroughly with warm tap water (40°C approx.).

method C: Sanitize the accessories E2, E7, E8 and E9 by boiling them in water (temperature $\geq 90^{\circ}\text{C}$) for 20 minutes (except for the connection tube) preferably using distilled or demineralised water to prevent lime scale deposits.

After sanitizing the accessories E1, E2, E3, E7, E8, E9, F1, F2, F3, F4 shake them vigorously and lay them on a paper towel, or dry them with a jet of warm air (i.e. a hair-dryer).

Disinfection

The accessories that can be disinfected are E1 without elastic E2, E3, E7, E8, E9, F1, F2, F3, F4

The disinfection procedure described in this paragraph should be carried out before using the accessories. This procedure is effective on the components only if each step is carefully followed and if the components are previously sanitized.

Execution:

- a) Use a container of a suitable size to contain all the components to be disinfected. Fill it with a solution of potable water and electrolytic chloroxidizer (active ingredient sodium hypochlorite) suitable for disinfection according to the dilution proportions recommended in the disinfectant instructions.
- b) Submerge every component completely in the solution; try to prevent that air bubbles form in contact with the components. Keep the components immersed for the period recommended on the disinfectant instructions based on the selected concentration / dilution.
- c) Remove the disinfected components and rinse them thoroughly under warm potable water. Dry the components as described in the Sanitization paragraph. Discard solution according to the instructions provided by the disinfectant manufacturer.

Air filter

The device has an inlet filter (Pic. B4) that should be replaced when it is dirty or changes color. Do not wash or reuse the same filter. Regular replacement of the filter is required to help and ensure correct performance of the compressor. Check the filter on a regular basis. Contact your local dealer or authorized service centre for spare filters.

Use original manufacturer accessories only.

Re-use

Before re-use the compressor has to be disinfected by wiping it with a damp cloth with an antibacterial detergent (non abrasive and solvent free) which is compatible the material of the compressor. The material of the compressor is ABS (Acrylnitril-Butadien-Styrol).

The filter and the nebulizer set (C6, E1, E2, E3, E5, E6, E7, E8, E9) have to be exchanged.

Symbols

Class II unit

Type-BF applied parts

Important: check the operating instructions

Alternate current

• BAUART
• GEPRÜFT
• TYPE
• APPROVED

TÜV Certification ref. EN 60601-1 3rd Ed.

Switch OFF

Switch ON

Hazard: electrocution.
Consequence: Death
Do not use device while
taking a bath or shower

In compliance with European standard EN 10993-1 Biological evaluation of medical devices and 93/42/EEC Medical Device Directive

IP21

Enclosure protection rating: IP21.
(Protected against solid bodies over 12 mm.
Protected against access with a finger; Protec-
ted against vertically falling water drops.)

Technical data (Mod. P1207EM F1000)

Voltage:	230V~ 50Hz 140VA
In conformity with:	Dir 93/42/EEC, CE0051
Max Pressure:	2.6 ± 0.4 bar
Compressor air output:	10 LPM approx.
Noise level (at 1 m):	54 dB (A) approx.
Operation:	Continuous use
Dimensions (W)x(D)x(H):	170 mm x 200 mm x 105 mm
Weight:	1.200 kg

Applied parts

Type BF applied parts include: (E1, E2, E3, E7, E8, E9)
 E1-Soft child mask: PP*+TPE* · E2- elbow adapter: PP
 E7, E8, E9 silicone masks: silicone
 E3- Mouthpiece with valve: PP*+ silicone · E5- RF7 plus nebulizer: PP* + silicone
 C6- Connection tube: PVC phthalates free

RF7 plus nebulizer for children

Medication minimum fill volume:	2 ml
Medication maximum fill volume:	8 ml
Operating pressure (with neb.):	1.00 bar approx.
Delivery rate:	Min With valve Max
data measured according to Flaem I29-P07.5 internal procedure	0.23 ml/min 0.53 ml/min

In vitro testing certified by TÜV Rheinland LGA Products GmbH – Germany in compliance with the European Standard EN 13544-1 for nebulising systems. Values gathered with mouthpiece. Further details are available upon request.

	Min With valve	Max
MMAD:	2.72 µm	2.53 µm
Breathable fraction < 5 µm:	77.8 %	79.6 %

Operating conditions:

Temperature:	min. 10 °C; max. 40 °C
RH Air humidity:	min. 10 %; max. 95 %

Storage conditions:

Temperature:	min. -25 °C; max. 70 °C
RH Air humidity:	min. 10 %; max. 95 %
Operating/storage atmospheric pressure:	min. 69 kPa; max. 106 kPa

* Polypropylene ** Thermoplastic elastomers

Service and maintenance

Repairs must be performed by authorized personal only. Any unauthorized repairs will void the warranty and may pose a safety hazard for the user. The MicroDrop® *Calimero2* is maintenance free.

Disposal

In conformity with Directive 2002/96/EC, the symbol shown on the appliance to be disposed of indicates that this is considered as waste and must therefore undergo "sorted waste collection". The user must therefore take (or have taken) the above waste to a pre-sorted waste collection centre set up by the local authorities, or else give it back to the dealer when purchasing a new appliance of the same type. Pre-sorted waste collection and the subsequent treatment, recovery and disposal operations favour the production of appliances made of recycled materials and limit the negative effects of any incorrect waste management on the environment and public health. The unlawful disposal of the product by the user could result in administrative fines as provided by the laws transposing Directive 2002/96/EC of the European member state in which the product is disposed of.

Electromagnetic compatibility

This device has been designed to satisfy requirements currently required for electromagnetic compatibility (EN 60 601-1-2:2007). Electrical medical devices require special care. During installation and use with respect to EMC requirements, it is therefore required that they be installed and/or used according to the manufacturer's specification. Potential risk of electromagnetic interference with other devices, in particular with other devices for analysis and treatment. Radio and mobile telecommunications devices or portable RF (mobile phones or wireless connections) may interfere with the operation of electrical medical devices. The manufacturer reserves the right to make technical and functional changes to the product without notice.

For additional information please visit www.mpvmedical.com.

Order details

Designation	REF	PZN
MicroDrop® Calimero 2 Professional inhalation system for children 0 – 6 years , complete with compressor, RF7 plus nebulizer, soft child mask, silicone masks size XS, S, M, mouthpiece, air tube, 1x spare filter, carrying bag, instruction manual	M 51804-00	08628235
MicroDrop® Profi Nebulizer Kit children Universal RF7 plus nebulizer, mouthpiece with expiration valve, air tube (can be connected also to other compressors) soft child mask, (Phtalate- and DEHP-free PVC), tube connector, manual instruction	M 53700-12	00347459
MicroDrop® Calimero 2 Year-Kit RF7 plus nebulizer for children, mouthpiece, air tube, 1x spare filter compressor MicroDrop® CalimeroJet, 1x spare filter compressor MicroDrop® Calimero2, instruction manual	M 53700-17	03071182
Accessories		
MicroDrop® RF7 plus Nebulizer for children with mouthpiece incl. valve, without air tube	M 53500-03	08628241
MicroDrop® RF7 plus Mouthpiece with expiration valve	M 50020-43200	00023001
MicroDrop® RF7 plus Soft Child Mask yellow/transparent with rubber band	M 50020-44200	00348476
Universal adapter (Tube Connector) to connect to other compressors	M 53400-00	03435974
MicroDrop® 2 Air Tube 2m with 2 fits-all-connectors (Phtalate and DEHP free PVC)	M 50020-42400	09937470
MicroDrop® RF Adapter / angle for babies with expiration hole to connect with Silicone Mask and Tracheostoma Mask	M 50020-43800	00347703
MicroDrop® 2 Compressor filter (1 piece spare filter)	M 50010-42700	09937518
MicroDrop® Calimero 2 Noozle for RF7 plus nebulizer	M 50020-44500	03954496
MicroDrop® Calimero 2 Instruction Manual	M 50010-55800	–
MicroDrop® RF7 plus Nebulizer for children Instruction Manual	M 50010-55900	–
MicroDrop® Calimero 2 Brochure	M 90010-33700	–

Masks

Baby angle / Angle adapter	Babys 0–12 month Silicone mask size XS	Kinder 1–3 Silicone mask size S	Kinder 3–6 Silicone mask size M	Soft Child Mask
				

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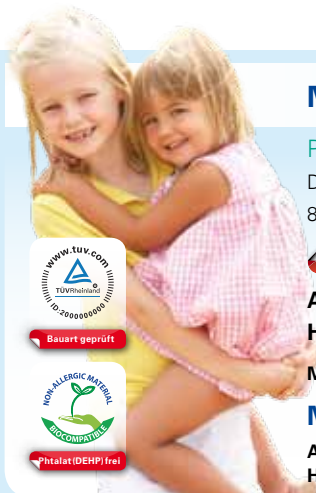
Warranty conditions

- The device is covered by a 5-year warranty as indicated on the gift box from date of purchase for any defects in the design or in the materials used.
- The warranty consists in the replacement and/or repair free of charge of originally defective components.
- The warranty does not cover the accessories supplied with the device and those parts subject to normal wear and tear.
- The device must be repaired by authorized technical service centres only.
- The device must be sent to the authorized service centre for repair within 8 days after the defect is noticed.
- Transport costs for the device are at user's charge.
- Any repair out of warranty conditions are at user's charge.
- The warranty does not cover failures resulting from misuse, abuse of the product or in case the damage is not due to manufacturer's liability (accidental fall, rough transportation, etc.).
- The warranty does not involve any compensation for any kind of damages to persons or things, either direct or indirect, occurred when the device is not operating.
- The warranty is valid from the date of purchase certified by the ticket or the invoice.

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MPV MEDICAL GmbH
FLAEM NUOVA S.p.A., Via Colli Storici, 221-223-225,