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INTENDED USE

The bath step shall provide easier access to the bathtub or shower stall or can be used for children to help them in reaching e.g. the washing basin. A single Bath Step has a height of appr. 97mm and is provided with an anti-slip surface. Each step is stackable (an additional 5cm of height per module) to form stairs or to increase the stepping height.

Mounting

- For Single Step: Place the caps (taped to the bottom side) in the open spaces in the bath step's top. Ensure that the Bath Step is clean and dry, remove the protective film from the back of the mat and place on top of the step and press firmly.
- For Multiple Steps: Place the additional step directly on top of the first step. The feet underneath the top step should align with the holes on the bottom step. Repeat for additional steps.
 - Place the caps (taped to the bottom side) in the open spaces in the top step's surface. Ensure that the Bath Step is clean and dry, remove the protective film from the back of the mat and place on top of the step and press firmly.
 - Note: If you stack several Bath Steps on top of each other, the anti-slip mat is only placed on the highest step. In this case, please keep the extra mats so you can replace the top mat later on (if it gets worn out or damaged).

⚠ CAUTION:

As always, use extreme caution when getting into or out of the bathtub at all times.
Maximum user weight: 418 lbs / 180 kg

Material

Bath step polypropylene
Support Mat Ethylene-vinyl acetate
Feet thermoplastic rubber

Technical data

weight 1,6 kg
dimensions 50 x 40x 10 cm
Step surface 45 x 35cm
Max. Load capacity 180 kg