# ActiveColor Rückenbandage

active  
COLOR

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**REF** 1490

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D1490I2023-10I005 ML I Rev. 01

### DE ActiveColor Rückenbandage

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### EN ActiveColor Back support

Many thanks for placing your trust in a medical device from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device.

#### Intended purpose

This medical device is a back support for the compression of soft tissue and slight stabilisation of the lumbar spine.

#### Indications

Back pain, lumbago, stress

#### Contraindications

Unclear soft tissue swellings, impaired sensation and circulation in the specific body region, skin diseases in the specific part of the body, latex allergy.

#### Application risks/Important notes

- The expert staff from whom you have received the medical device will select the appropriate size and instruct you regarding its use.
- remove the medical device prior to radiological examinations
  - remove it during longer periods of rest, e.g. when sleeping
  - in case of numbness, loosen the medical device and remove it if necessary
  - in case of persistent complaints, consult the physician or specialist retailer
  - use the medical device in accordance with therapeutic needs
  - only use other products simultaneously after consultation with your physician
  - do not make any changes to the product
  - do not wear it on open wounds
  - do not use in case of intolerance of one of the materials used
  - no re-use. This medical aid is intended for treating one patient only
  - whilst wearing the support, please neither use any creams nor ointments on or around it as they can destroy the material

#### Fitting instructions

Close the back support using the Velcro in the abdominal area. The support should sit tightly, but not constrain.

#### Removal

Open the Velcro fastening to remove the support. Close the Velcro fastener in order to avoid early wear to the Velcro surface.

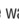
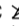
#### Material composition

Polyester (PES), elastodiene/latex (LA)

The sewn in textile label provides the precise material composition.

 The product contains latex and can trigger allergic reactions.

#### Cleaning information

 Delicate wash 30 °C  Do not bleach  Do not dry in a tumble dryer  Do not iron  Do not clean chemically  
Close the Velcro fasteners in order to avoid damaging other items of clothing. Do not use fabric conditioner. Stretch back into shape and dry in the fresh air.

#### Guarantee

The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for the indication, non-observance of application risks, instructions and unauthorised modifications to the product.

#### Useful life/Lifetime of the product

The lifetime of the medical device is determined by its natural wear and tear if treated appropriately and as recommended.

#### Duty of notification

If a serious worsening of the state of health occurs when using the medical device, you are obliged to notify the specialist dealer or us as the manufacturer and the Medicines & Healthcare products Regulatory Agency (MHRA). You can find our contact information in these instructions for use. You can find the contact information for the national authority responsible for your country under the following link: [www.bort.com/md-eu-contact](http://www.bort.com/md-eu-contact).

#### Disposal

Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

#### Declaration of conformity

We confirm that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL. You can find the current declaration of conformity under the following link: [www.bort.com/conformity](http://www.bort.com/conformity)

Status: 10.2023

 Medical device |  Single patient – multiple use

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