en

## INSTRUCTIONS FOR USE LILYPADZ NURSING PADS

**Packaging contains:** Two silicone nursing pads, two protective shells, storage case, instructions for use

**Indications:** Reusable nursing pads made from breathable silicone. The pads protect your breasts and clothes from milk leaking, helping you enjoy your freedom. LilyPadz can be used as an alternative to traditional nursing pads and are suitable for daily use. They are also ideal for swimming, wearing at night, evenings out and many other occasions. Discreet, reusable, dry.

**Contraindications:** Do not use if your nipples are sore or if you have thrush (fungal infection). Applying a lot of cream (e.g. Lanolin) to nipples can affect the adhesiveness of the LilyPadz, so only apply cream sparingly. A very strong letdown can also temporarily affect the adhesiveness of the pads. Contact your doctor if you experience irritation or redness on the skin.

### Usage:

- Always wash hands thoroughly before use. Breasts must be clean and dry before applying the LilyPadz.
- When using the LilyPadz for the first time, they should be cleaned as per the instructions.
- Remove the LilyPadz from the protective shells. Push the adhesive side of the LilyPadz (the inner side) outwards.
- Place the LilyPadz centrally over your nipple with the adhesive side against your skin and carefully press down onto your nipple (**important**).
- Smooth out the LilyPadz over your breast and you are ready to go!
- To nurse your baby, remove and clean the LilyPadz before placing them in the protective shells and storing them in the case.

**Cleaning instructions:** Always wash the LilyPadz after use using **hot water and** a pH-neutral soap (e.g. a liquid hand wash which does not contain lotions or oils). Every now and then the LilyPadz (without the protective shells) can be boiled (max. 1 minute). **The LilyPadz do not need to be boiled frequently and doing so will shorten the product life!** Do not wash them without using soap. Leave them to dry naturally. Do not place in the dishwasher or microwave. If cared for properly, LilyPadz nursing pads can be used for several weeks. Each pair is intended for use by one person only.

- **Tulips nipple shields are latex free.**
- **Disposal:** dispose of safely, according to local regulations
- **LilyPadz material:** silicone
- **Case material:** polycarbonate

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INSTRUCTIONS FOR USE
LILYPADZ® NURSING PADS

**Package list:** Two silicone nursing pads, two protective covers, a storage box, a user manual

LILYPADZ® Nursing Pads

**Adaptation range:** Reusable breathable silicone nursing pads. It can effectively protect your breasts and clothes from being stained by breast milk, and at the same time, it can also bring you a comfortable and free life. LilyPadz can replace traditional nursing pads, suitable for daily use. It is your ideal partner for activities such as swimming and night outdoor. Invisible, reusable, and keep dry.

LILYPADZ® Nursing Pads

**Contraindications:** Do not use on injured nipples and thrush (fungal infection) sites. Excessive use of nipple cream (such as lanolin) will affect the stickiness of LilyPadz, so it can only be used on the nipples. Apply a small amount of ointment. If the milk leakage reaction is very serious, it may temporarily limit its stickiness. If you experience skin itching or redness, please contact a professional.

LILYPADZ® Nursing Pads

- Usage method:**
  - Before each use, thoroughly clean your hands. Before placing LilyPadz, you must ensure that the breasts are clean and dry. Before the first use, please clean LilyPadz according to the instructions.
  - Remove LilyPadz from the protective cover. Turn the adhesive surface (inner side) of LilyPadz outwards.
  - Place the adhesive surface of LilyPadz in the middle of the nipple, and use your fingers to gently press the nipple into the breast (important).
  - Use your palm to press LilyPadz flatly on the breast—complete!
  - Before breastfeeding, please remove LilyPadz for cleaning, place it on the protective cover, and store it in the storage box.

**Cleaning:** After each use, please use hot water and pH neutral soap to clean LilyPadz. Occasionally, you can also boil LilyPadz (without protective cover) (no more than 1 minute). No need to boil frequently, because this will shorten the product's service life! You must use soap for cleaning. Do not dry it, and it must be naturally dried. Do not put it in detergent or microwave. If properly maintained, LilyPadz nursing pads can be used for several weeks. A pair of nursing pads is only allowed for one person to use.

- Processing:** Harmless, please follow the local regulations for disposal
- LilyPadz materials:** Silicone
- Storage box material:** Polycarbonate

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