MSA WaxExtract™

Deutsch

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MSA Wax Extract Ohrenreiniger

English

Instructions for Use
from page 11

MSA Wax Extract Ear Cleaner

Français

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Dear Customer,

we are pleased that you have chosen this product. If you have any questions, please contact customer service via our mail address:

kundenservice@hspfanshopping.de

Before using the product for the first time, please read the instructions carefully and use the Wax Extract as directed. Please keep the instructions in a safe place. If the product is passed on, these instructions must also be handed over. The manufacturer and importer accept no liability if the information in these instructions is not observed!


In the context of continuous further development, we reserve the right to change the product, packaging or instruction leaflets at any time.

Note

CE The CE marking confirms that the device complies with the applicable European requirements that apply to the respective device group and that the specific declarations of conformity are available.

SCOPE OF DELIVERY AND DEVICE OVERVIEW

- Ear cleaner with 9 button cells (LR44)
- 2 tips
- Cleaning brush
- Instruction manual

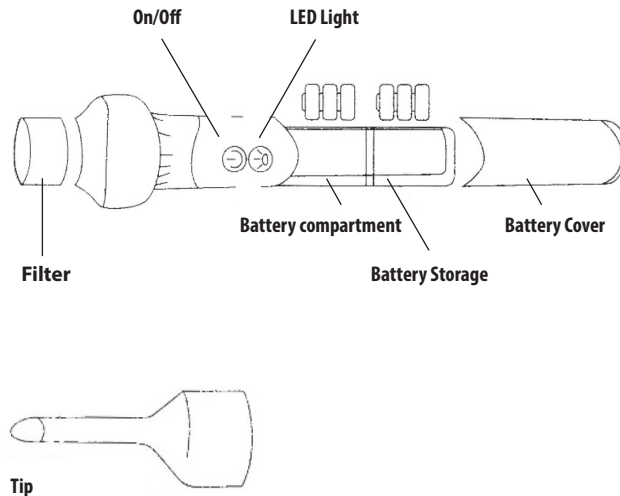GENERAL NOTES

- Before first use: Carefully remove the Wax Extract ear cleaner from the packaging and check the device for completeness and damage.
- This device is suitable for removing earwax in the human ear.
- This device must not be used by children and persons with limited knowledge of its operation or impaired sensory organs.
- Ensure that children do not play with the device.
- Cleaning and maintenance by the user must not be carried out by children.
- The device is intended for personal use only and not for commercial applications.
- Only allow authorized dealers to repair all types of damage to the device.

SAFETY INSTRUCTIONS

- Keep packaging material away from children and pets. There is a danger of suffocation.
- Risk of burns from battery acid! Keep battery and articles out of reach of children and animals. If a battery has been swallowed, medical assistance must be sought immediately.
- If a battery leaks, avoid physical contact with the battery acid. If necessary, wear protective gloves. In case of contact with battery acid, immediately rinse the affected areas with plenty of clean water and seek medical attention immediately.
- Immediately remove a leaking battery from the article. Clean the contacts before inserting a new battery.
- Only use the battery type specified in the technical data.
- Remove the battery from the article when it is used up or the article is not used for a longer period of time. This will prevent damage due to leakage.
- Protect the item from heat, open fire, direct sunlight, extreme temperatures, moisture and shock.

- Do not clean the item with water and never immerse it in liquids. It is only protected against splashing water.

WARNINGS

- Never try to remove excessive or hardened earwax from the ear with a paper clip, cotton swab or hairpin (for example). The earwax could be pushed further into the ear canal and cause serious damage.
- Never press the tip of the product too hard or too deep into the ear.
- Do not use ear drops if you suspect an ear infection unless a doctor recommends them.

Caution!

Consult a physician before use if any of the following apply to you:

- Ear pain
- Itching or rash in and/or around the ear
- Dizziness
- Injury, wound or perforation of the eardrum
- Recent operations on the ear

In case of pain in the ear, please consult a doctor.


When using the device, please sit in a safe place.

When removing wet earwax, please be careful not to clog the tip and filter.

INSERTING THE BATTERY


1. Pull the battery cover downwards. (Figure 1)

Figure 1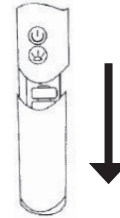

2. Insert 3 button cells (LR44) into the battery compartment of the main unit. (Figure 2) Ensure correct polarity. The rear compartment is for storing 3 more button cells.

Figure 2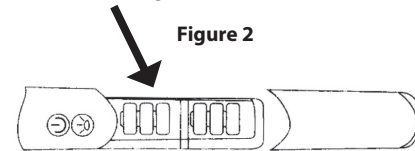

3. Push the battery cover back into place.

OPERATION AND APPLICATION

Place a tip on the main body. (Figure 3)


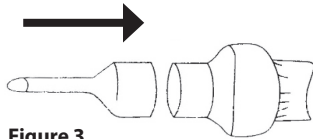
Figure 3

Press the ON/OFF key for operation. When the LED illumination is needed, press the ON/OFF key for the LED illumination.

To switch off, press the ON/OFF key again.

Always hold the product in a horizontal position (as shown) with the opening facing downwards.

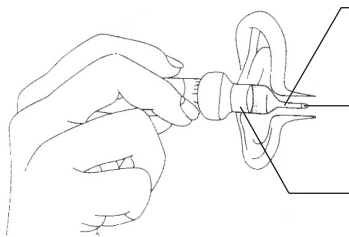Gently insert the tip into the ear. Repeat if necessary.

To ensure that the ear is not injured, never press too hard or too deeply into the ear.

CLEANING AND MAINTENANCE

1. Switch off the product before cleaning.
2. Clean the tips with warm water after each use and let them air dry.
3. The filter can be removed and cleaned with water and dried in the air. To do this, pull the filter out of the main body.
4. NEVER immerse the device in water.

TECHNICAL DATA

Battery type: LR44

DISPOSAL

In connection with the batteries and rechargeable batteries placed by us on the market or with the delivery of devices that contain batteries or rechargeable batteries, we are required by the Battery Act (BattG) to inform you of the following: The symbol of the crossed-out garbage can means that batteries and rechargeable batteries must not be disposed of in household waste. If the battery or accumulator contains more than 0.0005 mass percent of mercury, more than 0.002 mass percent of cadmium or more than 0.004 mass percent of lead, the symbol outside the crossed out garbage can is extended by chemical symbols of the metals (Hg, Cd, Pb). As an end user, you are legally obligated not to dispose of all used batteries and accumulators in household waste, but to return them to retailers or municipal collection points. Used batteries and accumulators that are not enclosed in the old device, as well as lamps that can be removed from the old device, must generally be separated from the old device in a non-destructive manner before being handed in at a collection point. Used batteries and accumulators may contain harmful substances that can damage the environment or your health if not stored or disposed of properly. Returned used batteries and accumulators will only be recycled. Thus, valuable raw materials such as iron, zinc, manganese or

nickel are recovered. You can either return used batteries or accumulators free of charge in your immediate vicinity (e.g. to a retailer or recycling center) or send them back to us free of charge. The environment and HSP Hanse Shopping GmbH thank you.

The symbol of the crossed-out dustbin that this device is subject to the EU Directive on Waste Electrical and Electronic Equipment (WEEE Directive). It indicates that the respective device must be disposed of separately from unsorted municipal waste at the end of its service life. A black bar under the crossed-out wheeled garbage can indicates that the device was placed on the market after 2005. Do not dispose of this device as normal household waste, but collect it separately and dispose of it in an environmentally sound manner at a municipal collection point. Owners of old appliances from private households can hand them in free of charge at the collection points of the public waste disposal authorities or at the take-back points set up by manufacturers or distributors as defined by the Electrical and Electronic Equipment Act. Stores with a sales area of at least 400 m² for electrical and electronic equipment are obliged to take back the equipment. From 01.07.2022, those grocery stores with a total sales area of at least 800 m² that offer electrical and electronic equipment several times a year or permanently and make it available on the market are also subject to this requirement. This also applies to sales using means of distance communication if the storage and shipping areas for electrical and electronic equipment are at least 400 m² or the total storage and shipping areas are at least 800 m².

The recycling symbol consists of three arrows representing the reuse cycle (recycling). The packaging and filling materials can be recycled. Do not dispose of the packaging and filling materials in the household waste, but take them to the recycling collection in an environmentally friendly manner.

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HSP Hanse Shopping GmbH • Im Hegen 1 • 22113 Oststeinbek • Deutschland
HSP Hanse Shopping GmbH • Im Hegen 1 • 22113 Oststeinbek • Germany
HSP Hanse Shopping GmbH • Im Hegen 1 • 22113 Oststeinbek • Allemagne
HSP Hanse Shopping GmbH • Im Hegen 1 • 22113 Oststeinbek • Duitsland